*de Luxe*

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**RIDDER GmbH**

Ridder Straße 1, D-56379 Singhofen  
Tel.: +49 (2604) 9541-0, Fax +49 (2604) 9541-50  
Mail: [info@ridder-online.de](mailto:info@ridder-online.de), [www.ridder-online.de](http://www.ridder-online.de)

Art.-Nr.: A0300100

# Rollator

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Instruction Rollator/Walking frame A0300100

Dear Customer,

Thank you for choosing our product. Please read the safety instructions carefully before use! The manufacturer assumes no liability for improper use of the Rollator /Walking frame.

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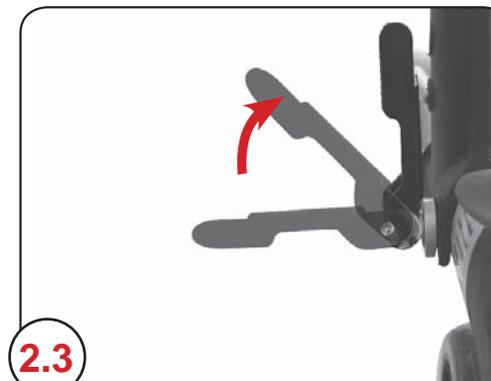
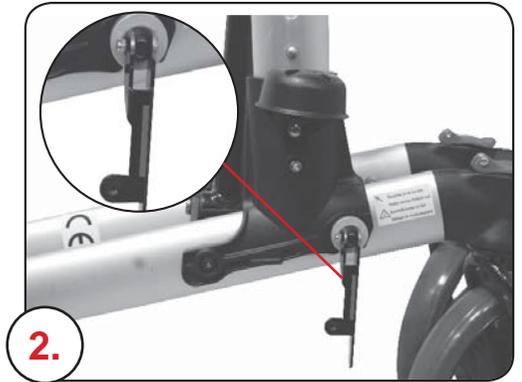
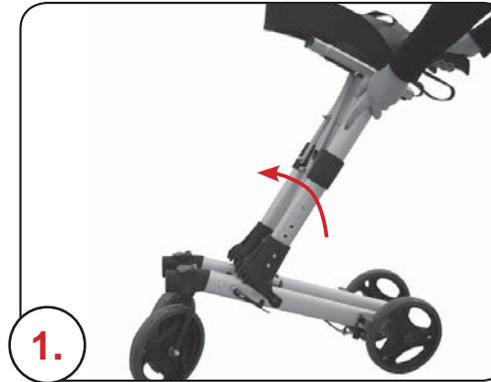
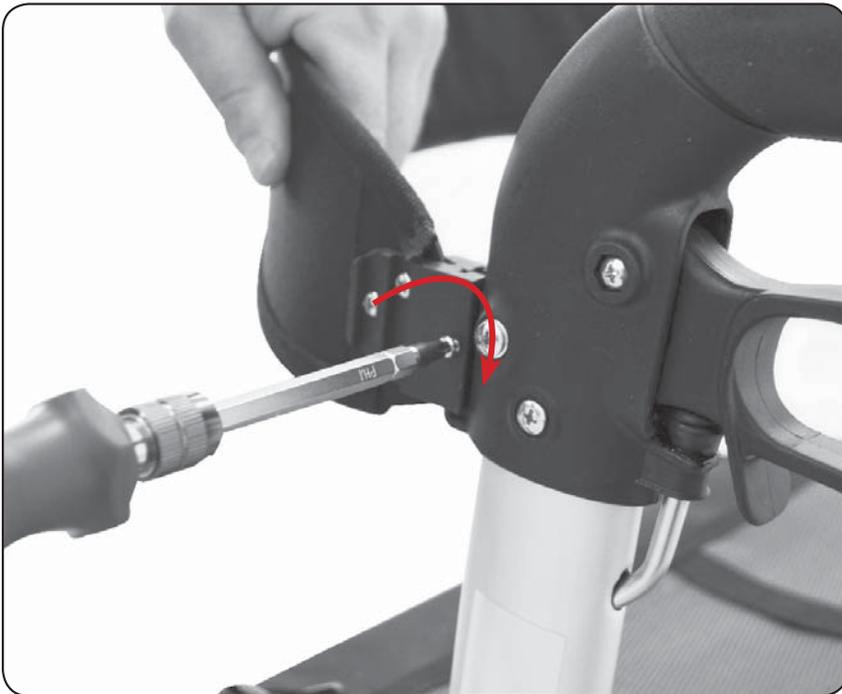
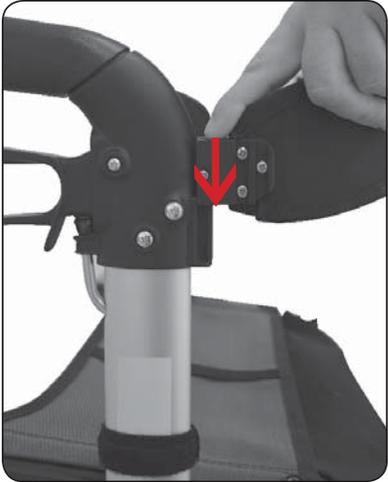
### SAFETY SUMMARY

DO NOT install or use this equipment before reading and understanding of this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.

Each individual should always consult with her/his physician or therapist to determine the proper adjustment and usage.

Rollators/Walking frames are NOT intended to be self-propelled while seated.

- Always check if the seat tube is locked and well connected to the frame when the Rollator/Walking frame is open.
- Always test the break after every height adjustment in order to avoid a free flex of the cable.
- Always test, whether the both of the handgrip rails are secured at the same height and the snap buttons for the handgrip height adjust are fixed correctly (are out)

Please check before each use, whether the front steering wheels are engaged properly

Failure to follow these warnings can result in personal injury.

All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure the Rollator/Walking frame is properly balanced.

The brakes MUST be in the locked position BEFORE using the seat.

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**Important:** Check all parts for shipping damage. In case of shipping damage, DO NOT use the product, contact the Dealer/Carrier for further instructions.

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### stability warnings

The Rollator/Walking frame can provide ambulatory assistance to an individual weighing up to 136kg, INCLUDING the loading of the bag. The Rollator/Walking frame bag has a weight limitation of 4.5kg